## Nederlands – العربية – Arabisch

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Ihr OPED-Team

Please read the instructions for use and safety advice before using the Pollex. You will find readable instructions for use on our OPED homepage.

Intended purpose: thumb fixing to immobilise the carpometacarpal and thumb base joints.

Safety advice

- The product is single use and should only be used by one patient.
- Never apply the product to open wounds and only use under prior medical instruction.
- The patient may only adjust it in consultation with his/her treating doctor.
- Should you experience excessive pain, swelling, numbness or sensitivity changes, unusual reactions or an uncomfortable sensation while wearing the device, please contact your doctor or orthopaedic technician immediately.
- This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
- Please consult your treating doctor before using it in conjunction with another medical device.
- Only replacement parts approved by OPED GmbH may be used.
- Please observe the road transport law of your country.
- Latex-free

Note: Not made from natural rubber latex

How to use (instruction video can be found at app.oped.de)

- Open all of the Velcro fasteners on the Pollex (Figure 1)
- Insert the hand into the orthosis (Figure 1 A/B)
- Perform the fingertip grip (the thumb and index finger tips touch) (Figure 2 A)
- Close the Velcro fasteners on the thumb. Please note the sequence and start with the middle tab (Figure 2)
- Close the wrist strap (Figure 2 B)
- Tighten the Velcro fasteners if necessary
- Any excess material at the end of the wrist strap can be trimmed if needed (Figure 3 A)

Information regarding care / cleaning / disinfection / sterilisation

You receive the medical product in a hygienically perfect condition. Can be cleaned at 30 °C Drying: Air dry, do not tumble dry. Disinfection/Sterilisation: The medical product is not suitable for disinfection/sterilisation.

Product characteristics

Available in two sizes for the right and left hand. Size guide: Size S Wrist circumference up to 15,5 cm Size M/L Wrist circumference from 15 cm

Indications

- Ligament injuries of the thumb
- Immobilisation of the thumb
- Rhizarthrosis without surgical indication

Contraindications

There are no known contraindications.

Technical data/parameters

Hard shell (PE); textile (100% polyester); PU foam; straps (PA/PP)

Transport conditions / storage conditions / disposal

Store in a dry and clean place and protect from direct sunlight.

Please dispose of the medical device in residual or household waste.

Warranty

The warranty complies with statutory regulations. The terms and conditions of business of OPED GmbH also apply, where permissible.

Data protection

Germany: Information about the processing of your personal data can be found under: <https://oped-international.com/privacy-policy/>.

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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- Solo se pueden utilizar piezas de intercambio y piezas de recambio originales de OPED GmbH.