SUPROhand

Gebrauchsanweisung Instructions for use Mode d'emploi Istruzioni per l'uso

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Ihr OPED-Team

English

Please read the instructions for use and safety advice before using the SUPROhand. You will find readable instructions for use on our OPED homepage.

Intended purpose: A stabilising wrist orthosis with a volar and dorsal reinforcement splint along with a circular stabilisation band.

Safety Advice

- The product is intended only for use on a single patient.
- Never wear the product over open wounds and only wear it after prior medical instruction.
- The patient may only adjust it in consultation with his/her treating doctor.
- If excessive pain, numbness or sensation changes occur, or if unusual reactions or an unpleasant feeling are experienced while wearing the device, please contact your doctor or orthopaedic technician without delay.
- This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
- When used in combination with another medical product, consult your doctor.
- Only replacement parts approved by OPED GmbH may be used.
- Please note the Road Traffic Act applicable in your country.
- Latex-free ❌
- Not made from natural rubber latex.

Preparation of the orthosis

Wrist orthosis for bilateral use. With reinforcement splint for the outside (dorsal) and inner side (volar) of the hand. It will be delivered set up for use on the right hand!

Left/right

- The printed L (left hand) and R (right hand) show you the positions for putting on the SUPROhand.
- When putting it on the right hand (R symbol), the hand has the thumb on the left.
- When putting it on the left hand (L symbol), the hand has the thumb on the right. (Fig. 2)

Turning the reinforcement splint to change from use for right hand to left hand

By turning the wrist orthosis by 180°, the SUPROhand can also be used on the left hand. The reinforcement splint on the inside of the hand must be turned to the other side accordingly.

- Open the palm tab. (Figure 4)
- Pull the reinforcement rail all the way out of the holder and turn it around 180°. (Figure 5, 6 and 7)
- Note: the short curved side faces towards the fingertips when in the correct position.
- Guide the rail all the way back into the holder and close the tab. (Figure 8)

Note: The printing on the reinforcement rail must be visible. The arrows indicate the direction for pushing. (Figure 6)

- The reinforcement splint located on the back of the hand does not need to be changed when applying the orthosis.

How to Use (instruction video can be found at app.oped.de)

- Slip your hand into the SUPROhand wrist orthosis. (Thumb position figure 1+2)
- Pull the flexible straps with Velcro over the lower arm towards the body and around the wrist so it's as tight as required, then firstly fasten it to the corresponding Velcro strap on the side of the lower arm before fastening it all together.
- If needed you can also use the Velcro strap, adjust it and fasten it again.
- SUPROhand fits properly when the pre-formed, volar reinforcement splint is applied with a close fit to the lower arm, especially in the transition from wrist to metacarpals, with the hand in a neutral position (outstretched hand).
- If necessary, the reinforcement splint on the palm and back of the hand can be individually adjusted to the anatomical shape of the hand and forearm or in accordance with the physician's specifications by bending it.
- SUPROhand should sit on you hand in a fixed, stabilising position without applying uncomfortable pressure. (Fig. 3)

Information regarding care/cleaning/disinfection/sterilisation

You are receiving this medical product in a hygienically perfect condition. Can be cleaned at 30 °C

Washing: Remove the reinforcement splints, close the Velcro fasteners.

Drying: Air dry, do not dry in a tumble dryer.

Disinfection/sterilisation: This medical product is not suitable for disinfection/sterilisation.

Product characteristics

Can be used on both the right and left hand:
SUHA-01-1-1 Wrist diameter up to 17cm
SUHA-01-1-2 Wrist diameter over 17cm

Indications

Postoperative care of radial fractures, arthrosis of the wrist, following severe sprains, tendovaginitis, functional post-op after-care, arthropathy, interim care after the removal of plaster casts, treatment of initial stages of bone necrosis, drop hand.

Kontraindikationen

- Circulation problems or lymphatic soft tissue swelling
- Neurological disorders affecting the senses and trophic skin changes (sensory disorder with or without skin damage)
- Allergic, inflammatory or injury-related skin changes in the hand

Technical data/parameters

Material composition: Fabric (100% polyester), metal splint (aluminium)

Dimensions

Size: 19 x 10 cm
Weight: 0,07kg or 0,08kg

Transport conditions/storage conditions/disposal:

Store in a dry and clean place out of direct sunlight.

Disposal: Please dispose of the medical device in residual or household waste. There are no specific criteria for destruction.

Garantee

The guarantee is in line with statutory prescriptions. The terms and conditions of business of OPED GmbH also apply, where permissible.

Patient information

The product is made of raw materials that have successfully passed tests in accordance with OEKO-TEX Standard 100®.

Showering/hygiene: it is not recommended that you shower or wash while wearing the product.

Data protection

Germany: Information about the processing of your personal data can be found under: <http://oped.ch/kontakt/your-data/>

Switzerland: Information about the processing of your personal data can be found under: <http://oped.ch/kontakt/your-data/>

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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