Deutsch

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Ihr OPEd-Team

English

Please read the instructions for use and safety advice before using the VACOHand Flex. You will find a copy of the instructions for use on our OPEd website.

Purpose:
Orthosis for the after-care of flexor tendon injuries.

Safety information

- The product is intended only for use on a single patient.
- Never wear the product on open wounds and only wear after prior medical instruction.
- The patient may only adjust it in consultation with his/her treating doctor.
- Should you experience excessive pain, swelling, numbness or sensitivity changes, contact your doctor or orthopaedic technician immediately.
- This medical product has been developed for the indications listed and can be used for other accidents, and by liquids that are not compatible with plastics.
- When used in combination with another medical product, consult your doctor.
- Only replacement parts approved by OPEd GmbH may be used.
- Please note the Road Traffic Act applicable in your country.
- LateX free. ☒
- Not made from natural rubber latex.

Preparation (before putting on the patient)

● Piece	● Assembly key	● Capuchons de crampage
● Pump	● Finger tap	● Arm cover
● Spring attachment	● Shell	● Traverse de la main
● Nail band	● Hand strap	● Recouvrement du bras
● Spring balancer	● Velcro cover	

1. Position spring balancers (Fig. 1)
Attach the ends of the spring balancer to the velcro strap on the hand strap.

2. Fasten nail pad (Fig. 2)
Apply the nail pad to the anatomy of the hand (Fig. 10).
Fix the nail pad to the anatomy of the hand (Fig. 10).
Alternatively, we offer self-adhesive nail pads. Depending on the natural fat content of the fingernail, it may be necessary to change the nail pads once or several times.

3. Fasting the spring balancers to the fingernails: After placing the orthosis, connect the spring balancers with many conventional fingering rings.
4. Alternatively, we offer self-adhesive nail pads. Depending on the natural fat content of the fingernail, it may be necessary to change the nail pads once or several times.

2. Fasten nail pad (Fig. 2)
Apply the nail pad to the anatomy of the hand (Fig. 10).
Fix the nail pad to the anatomy of the hand (Fig. 10).
Alternatively, we offer self-adhesive nail pads. Depending on the natural fat content of the fingernail, it may be necessary to change the nail pads once or several times.

How to use (instruction video can be found at app-oped.de)

- Put on arm cover (material without any creases)
- Open the straps of the shell (Fig. 3)
- Prepare cushions if the balls are distributed unevenly: see "adjustment options - ventilation"
- Place the lower arm in the shell. (Fig. 4) The axis of rotation for the wrist and the elbow must match up. The finger base joints must match up with the angle of the finger tap. (Fig. 5)
- Adjust finger tap to the length of the hand by shifting the hand support where necessary
- Adjust the finger tap to the anatomy of the hand. (Fig. 6-9)
- Adjust the hand bridge in the palm of the hand (Fig. 10). It is also possible to adjust the hand bridge to the folds of the hand.
- Thread through the straps, starting with the hand strap, and close. (Fig. 13)
- For strap 1, position the traction direction again (thumb should retain free mobility)
- When wearing the thumb: Hook the spring balancer into the hole in the hand strap
- It is possible to register a resistance of the Equilibrer à ressort on a biased level of the traction strength of the spring balancer on the spring attachment may become misaligned (note direction of rotation). (Fig. 16)
- Fasten the straps again and check the angle setting.
- If necessary: Take off velcro cover and shorten the strap. (Fig. 17)

Adjustment options

Valve function

- The cushioning cushion will become soft - pull the valve ring upwards and slide side ways to allow air to flow in. (Fig. 18)
- The cushioning cushion will become hard - push the valve ring downwards and drawn vacuum with the pump. (Fig. 19)
- Close valve cap - maintain the level. (Fig. 20)

Joint setting

- Open and close the joint by using the key a quarter turn. (Fig. 21)
- The key can only be taken out if the joint is closed.

FIX angle settings (Fig. 22)

- Place the thumb and index finger together with the thumb and index finger and raise slightly
- Move the hand support and set the desired angle.
- Close the joint with the vacuum.

Liberte de mouvement ROM (Fig. 23)

- Place the thumb and index finger together with the thumb and index finger and raise slightly
- Move the hand support and set the desired angle.
- Close the joint with the vacuum.

Lengthening spring balancers (only for trained specialists) (Fig. 24)

- For the left hand support, insert the key into the left hole of the shell.
- Remove the spring attachment by turning it towards the arrow.

Inserting the spring balancer (only for trained specialists) (Fig. 25)

- How to use on the right hand: the cable comes out of the left-hand side of the spring balancer attachment (Fig. 25).
- How to use on the left hand: the cable comes out of the right-hand side of the spring balancer attachment (Fig. 25).
- Engage the cable into the hole of the spring balancer featuring the OPEd name into the spring balancer attachment (Fig. 26)
- Number of spring balancers is ressed and depending on the indication
- Put the top back on the support until it makes a "CLACK" sound

5-fingeres Versorgung

- Fasten the spring balancer for the thumb in the hole on the hand strap diagonally.
- The middle and ring fingers are connected together with an elastic band.
- Attach the spring balancer to the ring finger using fleeces.
- The spring balancer of the thumb is attached to the other spring balancers. (Fig. 27)

Information regarding care/cleaning/disinfection/sterilisation

You are receiving this medical product in a completely hygienic condition.
Only clean plastic parts and padding covers in the cloth washing machine at 40 °C.
Lavage: parties en plastique et rembourrages dans la machine à laver à 40 °C.
Umweltfreundlich: keine aggressive Reinigungsmittel verwenden.
Drying: Leave to dry naturally, do not tumble dry.
Disinfection: Use disinfectant with plastic-compatible disinfectant.
The medical device is not suitable for sterilisation.

Product characteristics
VACOHand Flex universal right
VACOHand Flex universal left
Recommended use for a wrist circumference of approx. 13 cm to approx. 20 cm.
Otherwise, at the discretion of the treating doctor.

Indications
Flexor tendon injuries

Contraindication
None known

Technical data/parameters
Materials used: Hard shell (POM); covers (synthetic); cushion covers (PVC). Filling material: (EPS-Kugeln); strap (PA material); spring balancers (steel/brass); hand support (PU foam/aluminium). All parts and components are latex-free.

Transport conditions/storage conditions/disposal
Store in a dry and hygienic place and protect from direct sunlight.
Germany: The medical device is a rental product and must be returned to OPEd.
Other countries: Please dispose of the medical device in residual or household waste.

Warranty
The warranty complies with statutory regulations. The terms and conditions of business of OPEd GmbH also apply, where permissible.

Patient information
Pressure points or pain
Inspect cushions for a crease-free fit.
Spread the cushion filling evenly at the pressure point.
Should pain or pressure points due to severe swelling not be alleviated despite several adjustments, go to your doctor immediately. You will need additional medical aftercare. You can find a number of tips and tricks at www.oped.de

Rehabilitation information
In after-care booklet provided

Data protection
Germany: Information about the processing of your personal data can be found under: <https://oped.ch/de/> (deklaration de protection des donnees)
In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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Information pour les patients
Pression points or pain
Inspect cushions for a crease-free fit.
Spread the cushion filling evenly at the pressure point.
Should pain or pressure points due to severe swelling not be alleviated despite several adjustments, go to your doctor immediately. You will need additional medical aftercare. You can find a number of tips and tricks at www.oped.de

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Protection des données
Suisse: Vous trouverez des informations sur le traitement de vos données personnelles à l'adresse: <https://oped.ch/fr/> (déclaration de protection des données)
In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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